

**STYLE I**

**LipoFill™ CANNULA, STYLE I**

CATALOG #'s: LS-DI7 7CM 16ga COLOR CODE: RED  
 LS-DI9 9CM 16ga COLOR CODE: RED



TIP DESIGN: BLUNT, COMPLETELY ENCLOSED, USED FOR REINJECTION

USE: Blunt tipped cannula without cutting action. Recommended where vessels may be prominent or large. Typically used in areas of nerves, vessels and glandular tissue, appropriate for hands or deeper tissue layers of the face.

NOTE: The LipoFill™ Aspiration (LS-DASP) cannula is used to harvest tissue for the Style I, II, or III cannula.

**STYLE I MINI**

**LipoFill™ MINI CANNULA, STYLE I**

CATALOG #'s: LS-DI3 3CM 19ga COLOR CODE: RED W/ GREEN OBTURATOR  
 LS-DI6 6CM 19ga COLOR CODE: RED W/ YELLOW OBTURATOR



TIP DESIGN: BLUNT, COMPLETELY ENCLOSED. USED FOR REINJECTION

USE: A 19ga needle with obturator for clearing clogged lumen. Blunt tipped as in all Style I, used in areas where nerves, vessels or glands may be an issue. A short length and narrow gauge allows maximum control for use around the eyes or other areas where control is important.

NOTE: The LipoFill™ Cannula, Style I, 9cm Cannula (LS-DI9) should be used to harvest tissue for the MINI Style I or MINI Style III injector. The parcel size of the LS-DI9 easily passes through the 19ga MINI cannulas.

**STYLE II**

**LipoFill™ CANNULA, STYLE II**

CATALOG #'s: LS-DII7 7CM 16ga COLOR CODE: WHITE  
 LS-DII9 9CM 16ga COLOR CODE: WHITE  
 LS-DII9C 9CM CONCAVE 16ga COLOR CODE: WHITE



TIP DESIGN: BLUNT, SLIGHTLY OPEN, USED FOR REINJECTION.

USE: The most commonly used reinjection cannula. Partially open tip passes through most tissue types, and its blunt design minimizes damage to underlying structure. It can be used around the lips, the malar eminence, cheeks, and other areas that do not require placement of tissue directly under the skin or in fibrous dense areas. The Concave design allows navigation around soft tissue structures and for special placement. Useful for enhancing the dorsum of the nose via the forehead, and through the glabella or nasion.

**STYLE III**

**LipoFill™ CANNULA, STYLE III**

CATALOG #'s: LS-DIII7 7CM 16ga COLOR CODE: BLUE  
 LS-DIII9 9CM 16ga COLOR CODE: BLUE  
 LS-DIII9 9CM CONCAVE 16ga COLOR CODE: BLUE



TIP DESIGN: FLAT, SPATULA, COMPLETELY OPENED, USED FOR REINJECTION

USE: Open tip design that provides for maximum control as the cannula is advanced through dense or fibrous tissue. Can be used to place tissue under the skin or next to the mucosa when injecting in or around the lips. The flat, spatulated tip aids in enhancing the border of the lips, the lower eye-lid and around the eye.

Additionally, the Concave design allows the surgeon to navigate soft tissue areas and assists in special placement.

### STYLE III MINI

#### LipoFill™ MINI CANNULA, STYLE III

CATALOG #s: LS-DIII3 3CM 19ga COLOR CODE: BLUE W/ GREEN OBTURATOR  
 LS-DIII6 6CM 19ga COLOR CODE: BLUE W/ YELLOW OBTURATOR



TIP DESIGN: FLAT, SPATULA, COMPLETELY OPENED, USED FOR REINJECTION

USE: A 19ga needle with obturator for clearing clogged lumen. The flat, spatulated tip provides maximum control and an easy advance to enhance the areas around the eye, especially directly beneath the skin in the lower eyelid, the lateral border of the eye, and around the lips and mouth. It is recommended for areas where it may be difficult to advance in fibrous tissues.

NOTE: The LipoFill™ Cannula, Style I, 9cm Cannula (LS-DI9) should be used to harvest tissue for the MINI Style I or MINI Style III injector. The parcel size of the LS-DI9 easily passes through the 19ga MINI cannulas.

### ASPIRATION

#### LipoFill™ ASPIRATION CANNULA

CATALOG #s: LS-DASP15 15CM 11ga COLOR CODE: BLACK  
 LS-DASP23 23CM 11ga COLOR CODE: BLACK  
 LS-DASP23C 23CM CURVED 11ga COLOR CODE: BLACK



TIP DESIGN: CANNULA USED FOR TISSUE HARVESTING

USE: The Aspiration canulas are used for harvesting tissue for reinjection. The special tip design with its blunt, yet open, style allows the harvested tissue parcel to flow through the standard Style I, II, or III tip design cannulas.

NOTE: The LipoFill™ Cannula, Style I, 9cm (LS-DI9) should be used to harvest tissue for the MINI Style I or MINI Style III injector. The parcel size of the LS-DI9 easily passes through the 19ga MINI cannulas.

### HOOD

#### LipoFill™ HOOD CANNULA

CATALOG #s: LS-DH1015 15CM 10ga COLOR CODE: ORANGE  
 LS-DH1023 23CM 10ga COLOR CODE: ORANGE  
 LS-DH1215 15CM 12ga COLOR CODE: GREEN  
 LS-DH1223 23CM 12ga COLOR CODE: GREEN



TIP DESIGN: HOOD TIP IS USED FOR REINJECTING TISSUE, ESPECIALLY USEFUL FOR TISSUE INJECTION IN AND AROUND BREAST IMPLANTS

USE: The Hood tip cannula is used for re-injection of large tissue volumes. Its "hooded" design enables safe injection into the breast for augmenting and around breast implants for post-operative refinement including:

- disguise the edges or rippling of existing implants
- disguise breast implants that do not look natural
- sculpt and shape the breast

### BODY

#### LipoFill™ BODY CANNULA, STYLE I

CATALOG #s: LS-DB1215 15CM 12ga COLOR CODE: RED  
 LS-DB1223 23CM 12ga COLOR CODE: RED  
 LS-DB1415 15CM 14ga COLOR CODE: RED  
 LS-DB1423 23CM 14ga COLOR CODE: RED



TIP DESIGN: BLUNT, COMPLETELY ENCLOSED, USED FOR GENERAL ASPIRATION

USE: A Style I tip design for general aspiration when performing soft tissue removal procedures. It can be used for harvesting tissue for reinjection in areas where large volume placement is required.